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
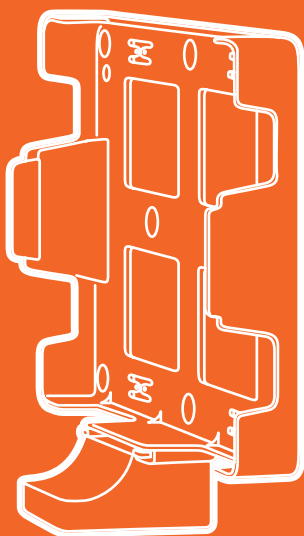
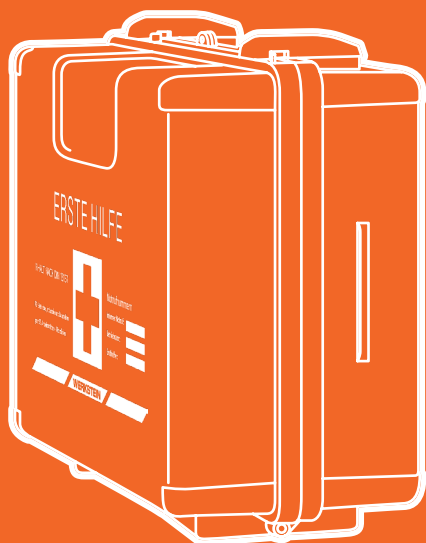
EN

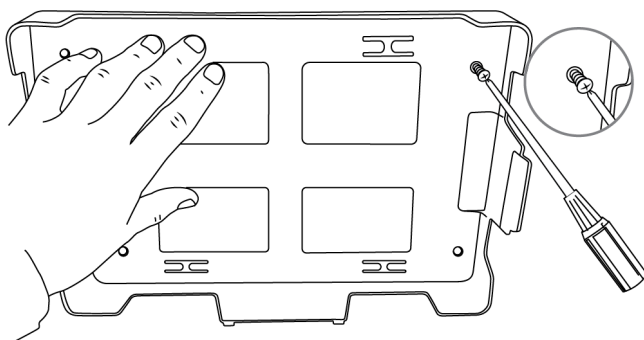- Remove the wall bracket from the case
- Hold the wall bracket against the wall with one hand and mark the locations of the holes (1-4) with a pencil in the bracket with the other hand. (Align side with the rail facing down)

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



EN

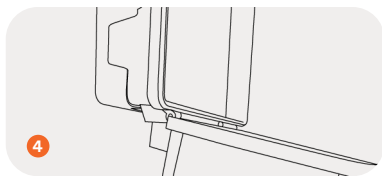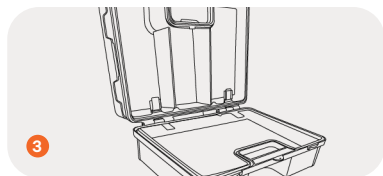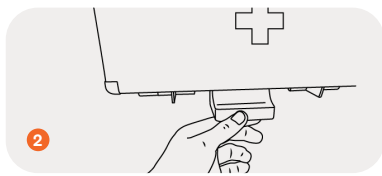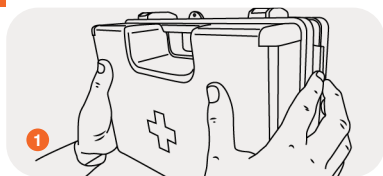- Drill holes in the wall at marked points
Insert dowels
- Tighten the wall bracket with screws

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

EN

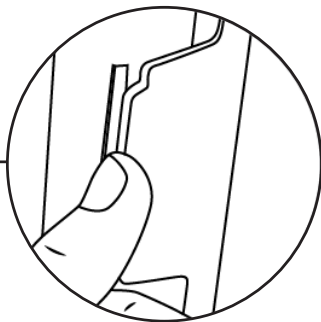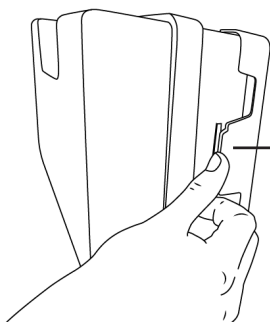- Insert the case into the wall holder.
- Optional for a 90° hold of the case: Insert the stopper into the rail at the bottom so that the case does not open completely downwards.

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EN

Taking out the case:

Hold the case by the handle with one hand and press down on the wall bracket with the thumb of the other hand. This makes the case easy to remove and mobile to use.

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EN

1. Safety seal for securing the first aid kit

The safety seal is used to securely close your first aid kit. It ensures that the contents remain untouched and ready for use until needed. To apply the seal, thread the end through the opening of the kit into the seal's closure until it clicks into place.

2. Safety stickers for behaviour in case of fire and emergency

The safety stickers provide important information for dealing with fires and emergencies. They should be placed in highly visible locations, ideally near the first aid kit.

3. Stickers with green directional arrows and first aid cross

These stickers serve to guide the way to the next first aid station. The sticker with the first aid cross should be placed directly next to the first aid kit. The stickers with the directional arrows can be placed along the path to the kit to clearly mark the route.

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Purawerk GmbH